

11 in 1 HydraFacial skin rejuvenation device

SPA08 User Manual



IMPORTANT: before starting the equipment correctly, please follow the installation instructions. Any damages caused by incorrect operations will void the warranty.

Safety Information:

Thank you for choosing our products! This user instruction is specially designed for the safety of you and others. Please read the instructions carefully before installing or using the machine and follow them when using.

Electrical Safety

Before installing the machine, make sure that your local electric network complies with the voltage and frequency indicated on the tag on the rear panel. Otherwise, the machine may be damaged in case of excessive current or voltage rise.

Never touch the machine with damp or wet hands or feet.

It is dangerous to change or try to modify the function of this machine in anyway.

Do not attempt to operate this unit if the power cord has been frayed or broken

Do not spill water or other liquids into or onto your unit.

During long periods of non-use, disconnect the unit's main power.

Safety Rules When Installing

Place the machine on a stable and secure base. It should not be subjected to any jolts or vibrations.

Never expose the unit to too high(>40°C) nor too low temperature(<5°C).

NEVER EXPOSE THE UNIT TO RAIN OR ANY OTHER HUMIDITY.

Do not expose the unit to direct sunshine or install the unit close to a source of heat(central heating, radiator...); otherwise the screen may be damaged.

Do not cover or block the ventil at or behind the unit.

Safety Rules When in Use

Well clean areas to treat before treatment.

Never touch the machine with damp or wet hands or feet.

Never place the cavitation handpiece on the liver, kidney area.

Operate Interface Instruction

Treatment screen interface.

Make sure the power line has been installed correctly; make sure the hand pieces are well connected with the equipment; ensure the circuit breaker (if any) at the back of the machine is pressed to "on". The working menu will appear automatically. Choose the Language. Choose the right parameter to do the treatment according to the needs of customer.

Turn on the machine and you can see the welcome screen and main selection interface.

1 . Instrument introduction

- 1.Hydrodermabrasion handle
2. Diamond dermabrasion handle
3. Ultrasound working handle
4. Three-polar RF handle
5. High Frequency handle
6. Skin scrubber



8. Spray mist gun
9. Photon brush (4 pieces)
10. 15 inch touch screen
11. Vacuum control knob
12. Water level knob
13. Power button switch



14. Monitor connection pug
15. Waste bottle
16. filter
17. Power socket
18. Power switch




*Before operating the machine, please check the connections. Then turn on and off the machine for 3 to 5 times. Then keep the machine running for 5 minutes to check the stability of the machine.

2 . Introduction of the Function

(1) Hydrodermabrasion

Hydradermabrasion– the latest in skin care technology. Hydradermabrasion utilizes the natural healing powers of water and oxygen to effort lessly exfoliate the skin without the use of hard crystals or abrasive textured wands, generating deeply hydrated and healthier looking skin.

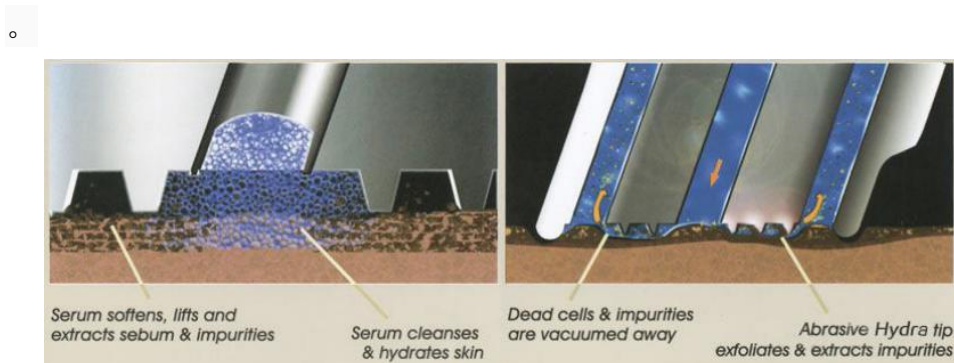

Application:

Rejuvenate sun damaged skin-face, neck, shoulders, back, arms and legs.

Reduce age spots.


Minimize of blotchy skin coloring.


Reduce acne and superficial scars from past injury.




Remove blackheads and whiteheads.

Reduce oily skin.

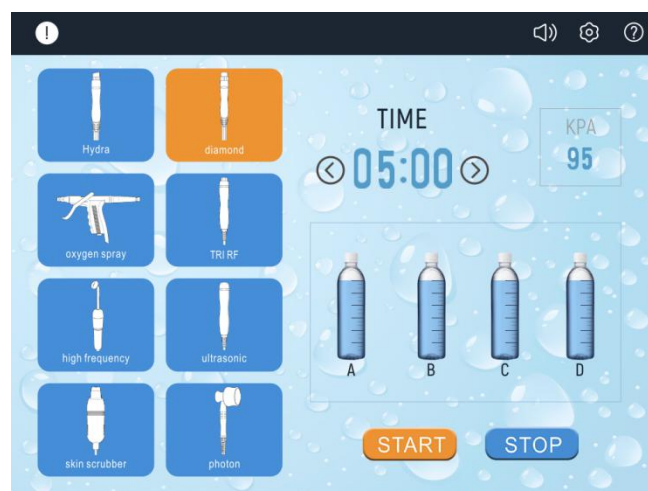Improve overall skin health.

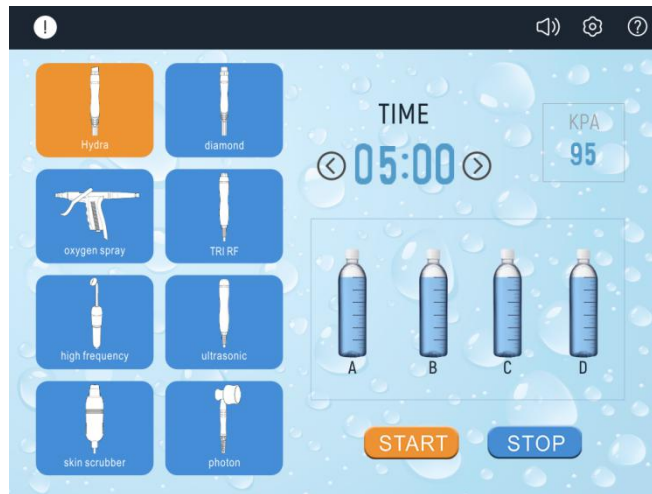



1. click  set working time , 1-60mins ;
- 2、choose bottle A/ B/ C/ D,
- 3、click  start operating ,  stop operating ,
4. When the set working time is up, the instrument will automatically pause,
5. Clean and disinfect the used working head.


Note: Please clean your face before use.

The suction and water flow can be adjusted through the two knobs on the front of the instrument.



(2) Spray gun operation interface

The nano-oxygen gun (deep water replenishment) sprays oxygen with nano-particles, and the water particles are fine, directly penetrating the bottom of the muscle, which is dozens of times the effect of ordinary spray. Atomized products, promote absorption, replenish water and oxygen.

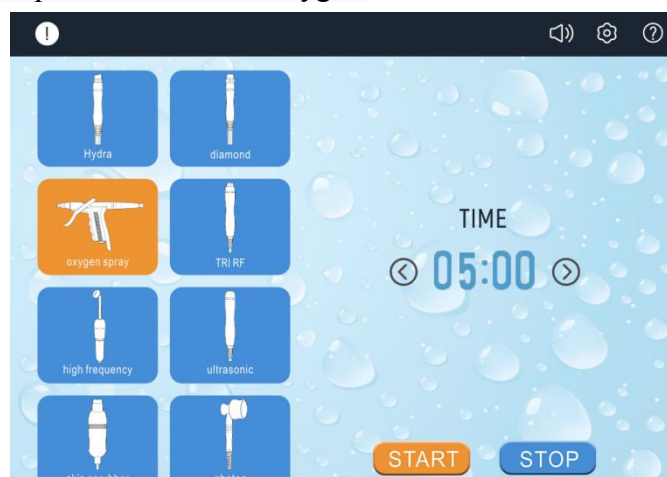

1. Pour the nutrient water into the small cup of the oxygen spray gun, the amount of water is about 2/3 of the cup, and then cover the lid. The small cup of the oxygen spray gun can only contain low-concentration non-sticky products. Products in other states are likely to cause pinholes to block.



1. click set working time , about 15-20mins,

3、click **START** start operating , **STOP** stop operating 。

Note: Clean the small cups and pipes in the sprayer after each use to avoid clogging.

Cautions:

- ① The product should be liquid when using oxygen sprayer.
- ② Avoid spraying into mouth or nose when spraying oxygen.
- ③ Clean the small cup and the pipe inside the sprayer every time use it, to avoid jam.

Clean method:


Pour the purified water into the small cup, and press the control button and then pullback towards.


Supply nutrients and moisture for the skin rapidly, smooth skin, lightening stains, improve chlorosis, dull conditions, make skin rosy, white, transparent, tender, tightening indeed, with special spa oils will be better.






(2) Three-pole radio frequency operation interface

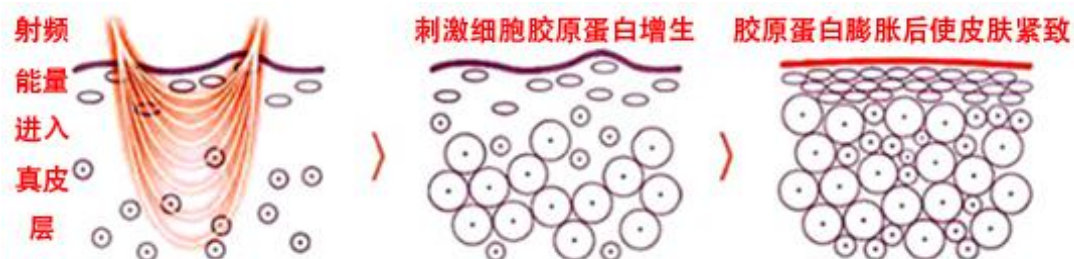(3)

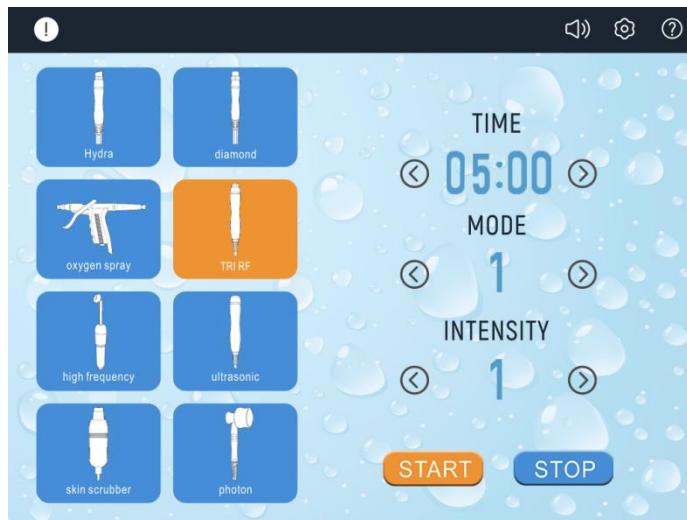The radio frequency wave penetrates the barrier of the epidermal basal melanocytes to heat the collagen fibers of the dermis to 55°C-65°C, and the collagen fibers shrink, so that the loose skin wrinkles are tightened, so as to achieve the purpose of beauty and wrinkle removal.






- 1、click  choose mode ,  adjust intensity ,
- 2、click  set working time , facial treatment time about 20-30mins,
- 3、 Apply gel to the care area, and move the electrode probe from the inside to the outside, from the bottom to the top. If there is less fat in the eyes and forehead, the energy output intensity should be appropriately reduced.
- 4、click  start operating ,  stop operating,
5. When the set working time is up, the instrument will automatically pause.
6. Clean the skin with warm water,
7. Clean and disinfect the used radio frequency work head.





Precautions:

1. The beautician should wear knotted gloves when operating.
2. The electrode probe must be in full contact with the skin.
3. The electrode probe is close to the skin to move slowly, not stop moving.

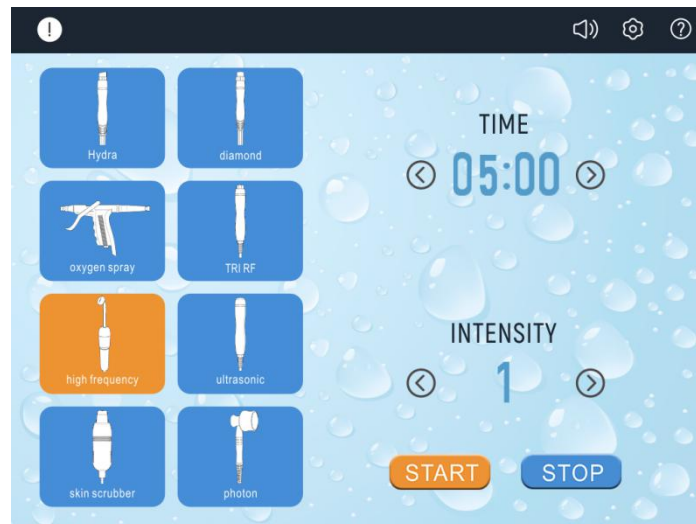(4) High Frequency

High frequency is through the high frequency electricity rate coagulation, urges the epidermal cell necrosis, which can instantaneously thoroughly eliminate all kinds of spots both on the face and on skin surface, such as old age spots, freckles, granules, tattoos, moles, etc. The effect is obvious with no bleeding or scar left.



- 1、click  set working time ;
- 2、click  adjust intensity ;
- 3、click  start operating,  stop operating 。

Operation steps:

① Indirect Method:

- ✧ Please screw in the glass tube in the hand piece.
- ✧ Please ask the client to hold the glass tube after touch some talcum powder. Turn on the machine and the electricity will get through from hands to the whole body.
- ✧ Beauticians do the massage on client's face from neck to chin and from visor to forehead.
- ✧ Please adjust the intensity to "0". Make sure that the machine is turned off and pull the glass tube out of the hand piece.

This method is fit for dry nature skin and aged skin.

② Direct Method:

- ✧ Please grease essence and cream on the face first and then the beautician holds the metal bar to put the glass tube on client's face. Turn on the machine.
- ✧ Massage with the glass tube like "Z" or helix on client's face, do the treatment as follows, forehead → nose → right side of the face → chin → left side of the face → nose → forehead.
- ✧ Please adjust the intensity to "0". Make sure that the machine is turned off and pull the glass tube out of the hand piece.

This method is fit for greasy nature skin.

③ Sparkle Method:

- ✧ Cover client's eyes with a piece of the wet cloth patch and touch the face with the glass tube on the part of inflammation. For one time, the treatment should be less than 10 seconds
- ✧ It's normal if there are sparkles come out while the glass tube touches the face.
- ✧ Please adjust the intensity to "0". Make sure that the machine is turned off and pull the glass tube out of the hand piece.

This method is fit skin with sore, inflammation and wounds

④ Hair Care Method:


- ✧ Use the comb tube together with pilatory.
- ✧ Toward the direction of the hair to move the comb tube.
- ✧ Please adjust the intensity to "0". Make sure that the machine is turned off and pull the glass tube out of the hand piece.

This method is for stimulating the skin surface, reducing folliculitis and improving the healthy growing of the hair.

Cautions:

- ① For sensitive skin, please cover a piece of voile to avoid the excess stimulation.
- ② Do not place on one part over 10 seconds.
- ③ Pregnant women and people with pacemaker are unfit for this treatment.
- ④ Please make sure that install the glass tube into the hand piece before turning on the machine. Adjust the intensity to "0" after finish.
- ⑤ Skin with freckle and splash is unfit for this treatment

(5) Ultrasound

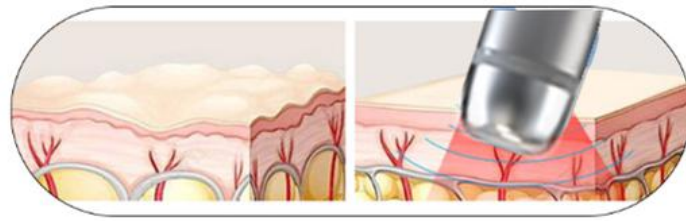




Ultrasonic, with the frequency of 1-3 million times per second, with the process to be transmitted to the body, is absorbed by epidermis dermis subcutaneous tissue, the absorption of ultrasonic waves is converted into thermal energy, thermal effects of ultrasound, the living tissue temperature increased slightly 0.5-1 °C, promote capillary function and fine inter cellular lymph flow, helps transport nutrients to the cells needed for blood vessels, promote metabolism and to assist the flow of lymph, maintain healthy skin. Maintain skin elasticity/ break down fat and prevent obesity. 1 million vibrations per second ultrasound, when the wave is transferred to the organization, with the ultrasonic vibrations stimulate biological tissue and skin tissue, relieve tension, to maintain skin elasticity. Heated generated by vibration can make it easier to break down fat and help prevent obesity.


Application:

*Melts fat and cellulitis cell, maintain skin texture smooth

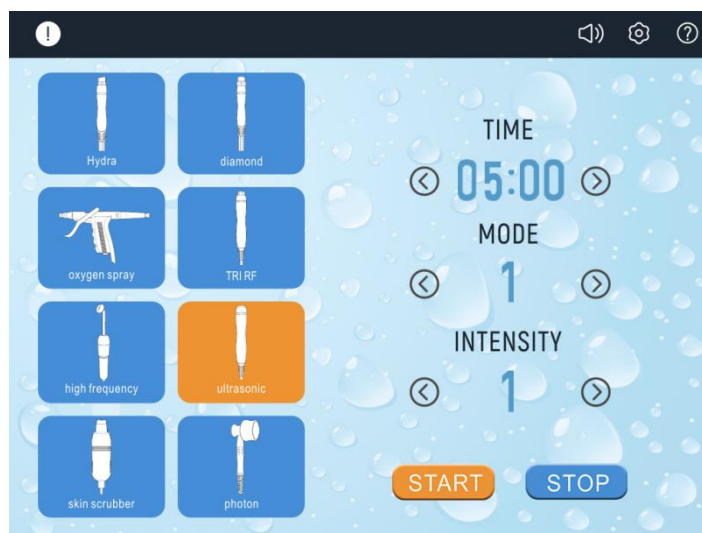

*Massage effect: helps to produce collagen and elastin, activates the fibroblast and enhances skin elasticity



1、click  choose mode , click  just intensity ,

2、click  set working time, facial treatment time about 20-30mins,

3、 Apply the medium (gel, essence) to the care area, and the dosage is subject to the flexible movement of the ultrasonic working head,

4、 click  start operating , click  stop operating,

5. When the set working time is up, the instrument will automatically pause.

6. Clean the skin with warm water,

7. Clean and disinfect the used ultrasonic working head.

***Precautions:**

1. Apply enough medium to the care area when operating.

2. Each part of the working hours should not exceed 30 mins.


3. Do not use on your heart etc.



4. Do not stay a long time in one part of the body.

5. After the instrument moves the audio work head can not be used for a long time in the no-load state

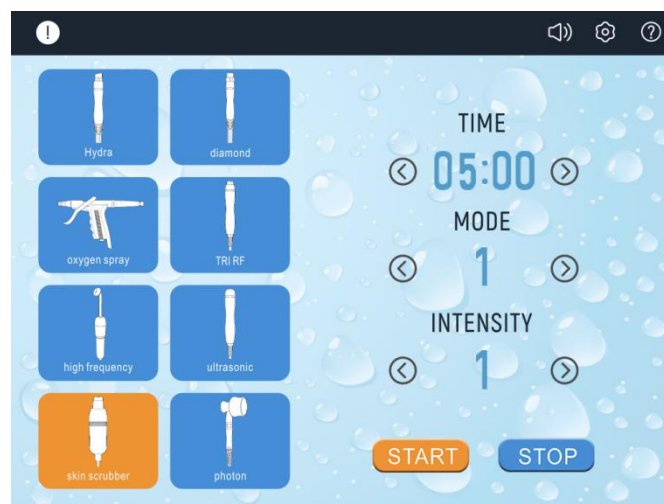

(6) Skin scrubber

With Ultrasonic Scrubber begins a new generation of aesthetic soft peeling. Using the high performance of ultrasounds, this exciting cosmetic breakthrough provides a safe painless and perfect physiological skin regeneration. By removing the superficial scaly layers of the skin, the Scrubber stimulates the revival of the skin tissue rendering it youthful and luminous from the first treatment.



1、 click  choose mode ,  adjust intensity ,

2、 click  set working time

3、 click  start operating ,  stop operating。

7. Photonlight (PDT)


General introduction of PDT:

Praised as “Magic light”, photon dynamical instrument is a biological active cold light, and not produce high heat. It applies low-energy and unique optical technology to skin care and treatment, natural and gentle; has prominent effects, low fatalness and no side effect. It's skin nurse specialist, is fit for all kinds of skin, especially useful for seriously irritated skin, acne skin and has not able effect for those in sub healthy state tore cover and do skin protection and treatment.

Photon dynamical nurse therapy is a new technology, using the photon power to do skin maintenance and treatment, transfer light energy to cell energy, then accelerate cell growth and blood circulation, stimulate fiber cells to produce collagen proteins, increase skin elasticity and take important part in rehabbing aging and acne skin, lightening flecks, skin upgrade and tighten and easing burnable skin.



Application:

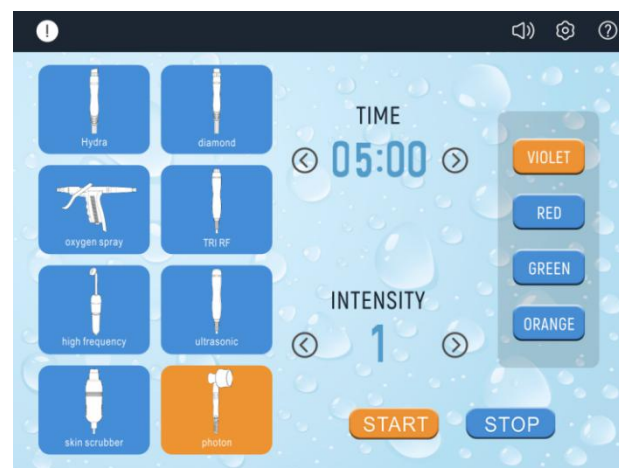

- ① Improve symptom such as aging and loosening skin, big pore, slender wrinkles.
- ② Improve pigmentary pathological changes, such as freckle, sunburn, senile plaques.
- ③ Improver dark complex ion caused by bad metabolism or poor circulation.
- ④ Repair and nurse the damaged skin.
- ⑤ Do detumescence, inflammation diminishing and scar elimination treatment for oil acne effectively.











- ① There are four colors which can be chose according to clients' need.



2. click  button to adjust intensity(from “1” to “9”). , Click  to set working time

- 3、click  to start operation , Click  to stop operation

3 、 Spare Parts

Hydrodermabrasion with 4 tips		Diamond dermabrasion with 4 tips/	
Ultrasound handle		Three polar RF	
High frequency handle		Skin scrubber	
Spray mistgun		Photon Brush 4 pieces	

3. Parameters:

Technology	Hydrodermabrasion/Spraymistgun/Ultrasound/Bio/Photoelectric/PDT
Vacuum	90Kpa
Power	300W
Max output	250VA
Ultrasound	1 MHZ
Screen	15' Touch screen
RF	5Mhz
High frequency	50Hz-60Hz
Handle	Hydrodermabrasion with 4 tips/Diamond dermabrasion with tip
	Spray mistgun 1piece; Three polar RF 1 piece;
	High frequency handle 1piece; Ultrasound handle 1 piece
	Skin scrubber 1 piece; Photon Brush 4 pieces
Voltage	100-240VAC,50Hz/60Hz
Warranty	1 years for main machine and 6 months for spare parts

5. Contraindications:

- (1) People who has fever, infectious diseases, acute diseases.
- (2) People with heart disease or configured cardiac pacemaker.
- (3) Patients with severe high blood pressure, tumor disease, asthma, deep vein thrombosis, varicose veins, thyroid, cancer, falling sickness.
- (4) People with hemorrhagic disease, trauma, vascular rupture, skin inflammation, skin Disease.
- (5) Pregnant women
- (6) Do not use at the abdomen during menstrual period
- (7) Medical Plastic parts, or parts with metal inside
- (8) People with an abnormal immune system

6. Product Service and Maintenance

- (1) Warranty: The warranty date is 2 years from the date of purchasing. We are responsible for the maintenance in our facility in GZ. When the warranty date is expired, we will charge the service fee or the cost of the component.
- (2) Professional Use: The merchandise of our Products is designed for professional use only and must be installed by a licensed professional electrician only. Failure to do so will void customer warranty.
- (3) Manufacturer Defects: In the event of defects immaterial and workmanship our obligation shall be limited to free labor and parts repair or replacement, solely a tour option, of any defective part for a period of one year from the date of purchase.
- (4) Disclaimer of Damages: The remedies provided here in are the buyer's sole and exclusive remedies and in no event shall our products be liable for direct, indirect, special, incidental, or consequential damages (including loss of profits)whether based on contract, tort, or any other legal theory.
- (5) Returns: All damaged machines must return in 30 days after receiving the confirmation from our service department.
- (6) Shipping Policy: For your protection, the acceptance of this shipment by the Transportation Company is an acknowledgement that the articles were delivered to them in good working condition and properly packed.
- (7) Exceptions
 - Inappropriate damage such as: dramatic impact, vibration, drop, condenser chamber burning, crystal rods destroying, etc.
 - Inappropriate damage or loss of key components such as the handpiece, protective glasses, power lines, you will be charged for the cost of accessories.

- Consumer who repairs the machine caused by improper by himself.
- Due to irresistible forces(such as fires, earthquakes and other natural disasters) of damage.
- Beyond the period of warranty products.